


00000160

RETURNING Gentech® Mo-99/Tc-99m Generators AUSTRALIAN CUSTOMERS

- 1) Refer to the calibration date on the Gentech® Generator and **ensure** minimum **4 weeks have passed from the calibration date**. The shipper of these return generators is responsible to ensure compliance.
- 2) Remove the needle from the generator Elution Port and rotate the top plate to cover the needle recess.
- 3) Remove or cover the radioactive labels from both sides of the packing carton.
- 4) Remove or cover the wording 'Radioactive material, Type A package UN2915' from both sides of the carton.
- 5) Affix a Radioactive material, Excepted package sticker on the outside of the packing carton. Generators must be returned as 'radioactive material, excepted packages, limited quantities of material - UN2910' .

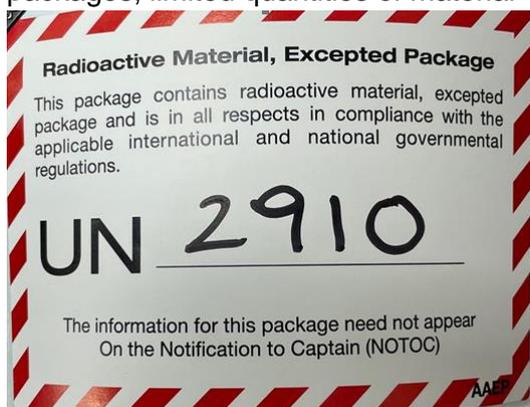

- 6) On the top of the packing carton, apply the Return Address Label over the original delivery label, ensuring the original barcode is covered or crossed out. Also ensuring any other barcode on the package is covered or crossed out with black marker so it cannot be read.
- 7) Place the generator into the packing carton and carefully replace the expanded polystyrene sections.
- 8) Seal the box with packaging tape.
- 9) For customers outside of the Sydney Metro a booking will need to be made with either TNT or Toll Priority.
 - (a) If using TNT, create an account online at www.mytnt.com, complete the shipping account details in order for a request to be sent to use ANSTO's account. Once approved an online booking can be made and the consignment note can be positioned and affixed to the packing carton.
 - (b) If using Toll Priority, request a consignment note from the ANSTO, Health Products Customer Supply Chain Team. Once received, complete the consignment note and contact Toll Priority (P: 13 15 31) to arrange collection.

10) All goods are to be returned by Road Transport Only

Consignee (Receiver)	ANSTO New Illawarra Road Lucas Heights NSW 2234
Charge Receiver	Cross this box
Contact Name	ANSTO Health Despatch
Contact Number	(02) 9717 9027
Description of Contents:	Endorse consignment note with the following statement: "Radioactive material, excepted packages –limited quantity of material – UN 2910"
Weight per Generator	23 kg
Dimensions	40 x 40 x 42cm
Number of Items returned.	The number of generators being Minimum per consignment is 4.
Service Required	ROAD FREIGHT /OFF PEAK ONLY
ANSTO Account Number TOLL PRIORITY 2008B5	TNT 90420581

Should you require any assistance or clarification please telephone the Customer Service Supply Chain team on 1800 251 572

**GUIDELINES FOR THE RETURN OF TECHNETIUM GENERATORS
AUSTRALIAN CUSTOMERS ONLY**

END OF DOCUMENT